

Thermo Wrap Product Line



TS226-UREST FOR KEYBOARD USERS



Repetitive strain injuries - RSIs - have become a workplace curse. RSIs can include everything from tendonitis to lower back strain.

Flexible Support

Wearer Comfort

Work or Play



TS204



TS205




TS208



TS209



TS212

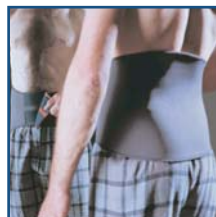

TS213



TS214



TS216



TS217



TS228



TS231



TS866



ISO 9001:2000

PO Box 524, 40 Dussek St., Belleville, ON, Canada K8N 5B2

Toll Free: (888) 232-0031 (N.America) 0800 0280 243 (UK)

Tel: (613) 966-0062 Fax: (613) 966-0067

www.2protect.com E-mail: impacto@2protect.com

Impacto™ is a registered trademark of Impacto Protective Products Inc. Trioxon® is a registered trademark of Spectrum Trading Ltd.

Thermo Wrap Product Line



TS 226

HOW TO FIT THE IMPACTO WRIST WRAP

Step 1
Place thumb through the hole with Trioxon lining (red) toward the skin.


Step 2
Pass over the back of hand and under the wrist. Take across the back of the wrist again and adjust tension to a comfortable level.

Washing Instructions: Hand wash in warm water using mild soap. Rinse well, line dry. Wrap must be washed every 2-3 days during normal use. Wash daily if worn during sports or heavy work.

CARPAL TUNNEL SYNDROME

Manual tasks during which the wrist is repeatedly flexed and extended can cause carpal tunnel syndrome.

(Tendon swelling in the wrist compresses the median nerve causing numbness, tingling or pain.)

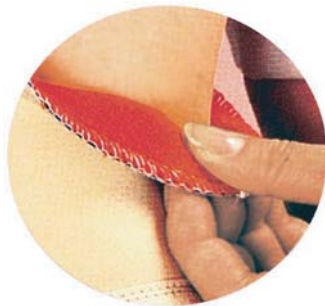

Alleviate Neoprene /Elastic Irritation with Trioxon® Lining

- Captures body heat
- Increases circulation
- Allows skin ventilation
- Wicks moisture
- Flexible support
- Provides compression

Carpal tunnel holds nine tendons, and the median nerve

Median nerve when pressed, results in loss of feeling in the fingers and hand

Tendon there are a total of nine

Sheath swells when irritated

Carpal ligament is cut in surgery to release pressure on the carpal tunnel. Of those with the syndrome, 85% don't need surgery

PO Box 524, 40 Dussek St., Belleville, ON, Canada K8N 5B2

Toll Free: (888) 232-0031 (N.America) 0800 0280 243 (UK)

Tel: (613) 966-0062 Fax: (613) 966-0067

www.2protect.com E-mail: impacto@2protect.com

Impacto™ is a registered trademark of Impacto Protective Products Inc. Trioxon® is a registered trademark of Spectrum Trading Ltd.

